

# NARK® II Color Identification Chart

Initiate progressive field testing with the Marquis Reagent, Test 01. Note color(s) and proceed to the next reagent indicated for that substance. Example; Heroin generates a purple in Test 01. Next, go to one of the confirming tests 02, 10 or 11. Heroin is presumptively identified in Test 11 after a dark green color is generated following the breakage of the right ampoule. Note: Squares without color indicate no color develops in that stage of the test.



NARK20016

TEST	REAGENT NAME	TEST FOR	FINAL COLOR
No. 1	Marquis Reagent	Amphetamines/Methamphetamine	Orange ► Brown (within 12 seconds)
		Heroin/Morphine	Orange ► Purple
		MDMA (Ecstasy)	Orange ► Black
No. 2	Nitric Acid Reagent	Heroin	Yellow ► Green-Yellow
		Morphine	Orange-Yellow ► Yellow
No. 3	Dille-Koppanyi Reagent	Barbiturates	Clear ► Lavender
No. 4	Ehrlich's Reagent	LSD (Lysergic Acid Diethylamide)	Clear ► Lavender ► Purple
No. 5	Duquenois-Levine Reagent	Marijuana, THC	Clear ► Purplish-Blue ► Light Blue-Purple over Dark Purple
No. 7	Scott Reagent (modified)	Cocaine HCl (powder)	Blue ► Pink ► Pink over Blue
		Cocaine Base (crack, freebase)	Blue Specks in Pink ► Pink ► Pink over Blue
No. 8	Methadone Reagent	Methadone	Blue
No. 9	PCP Reagent	PCP (Phencyclidine)	Blue Specks in Pink or Blue
		Methaqualone	Clear ► Blue
No. 10	Special Opiates Reagent	Codeine	Green
		Heroin, Morphine, Buprenorphine (Suboxone®)	Purple
		Oxycodone, Fentanyl	Yellow
No. 11	Mecke's Modified Reagent	Heroin (white, brown, black tar)	Clear ► Green
No. 12	Frohdes Reagent	Pentazocine	Purple ► Yellow
		DMT	Orange-Yellow ► Orange
No. 13	Ephedrine Reagent	Ephedrine/Pseudoephedrine	Clear ► Bluish-Purple
No. 14	Valium, Rohypnol, Methcathinone Reagent	Valium, Rohypnol	Clear ► Lavender
		Methcathinone	Clear ► Brown
No. 15	Sodium Nitroprusside Reagent	Methamphetamine, MDMA	Clear ► Clear ► Navy Blue
No. 19	Mayer's Reagent	Narcotic Alkaloids	Tan/Cream
No. 20	KN (Fast B Blue Salt) Reagent	Marijuana, THC	Clear or Tan ► Red-Brown
No. 21	GHB Reagent	GHB	Teal
No. 22	Mandelin Reagent	Methadone	Olive ► Dark Aqua
		Amphetamines	Green-Yellow
		Methamphetamine	Yellow-Green
No. 23	Synthetic Cannabinoid Reagent	(Synthetic cannabinoids) JWH-018; JWH-073; JWH-250; CP-47, 497; HU-210	Clear ► Yellow or Orange
No. 24	MDPV Reagent	Methylenedioxypropylvalerone; (Bath Salts)	Yellow or Green Yellow; if Clear, proceed to Test 25
No. 25	Mephedrone Reagent	4-methylmethcathinone (4-MMC)	Clear ► Purple
No. 26	a-PVP Reagent	alpha-pyrrolidinopentiophenone	Blue Specks in Pink ► Pink ► Pink over Blue
No. 27	Weber Reagent	Psilocybin (magic Mushrooms)	Red ► Dark Aqua
No. 29	2C Reagent	2C-B, 2C-C, 2C-3, 2C-I, 2C-N, 2C-T7	Yellow; Yellow-Green; Light Green; Brown; Red-Purple; Purple
No. 30	Psilocybin/Psilocin Reagent	Psilocybin	Pink
		Psilocin	Purple
No. 31	Liebermann Reagent	Amphetamine	Green
		Methamphetamine	Orange
		Morphine	Black
No. 32	Mollies Reagent	MDMA/MDMA Analogs	Orange ► Black
		2C/MDPV/Mephedrone	Yellow or Green-Yellow ► Clear
		Amphetamines/Meth	Orange ► Brown
No. 33	Fentanyl Reagent	Fentanyl (Acetylfentanyl)/Heroin	Orange; Purple
No. 34	Hemp/CBD Reagent	Hemp/CBD	Purple, Teal Blue